MEDICAL TRADING S.R.L.

MANUFACTURER OF AUTOCLAVE SINCE EVER, FOREVER

AUTOCLAVE

HYDRA 100

DIRECTIVE EN 13060

CLASS B

CLASS B - Big Small Sterilizers

Small autoclaves with similar performance to hospital autoclaves.

With these devices it is possible to sterilize any type of load: solids, bodies Cables (Type A and Type B), porous materials, textile loads, wrapped and unwrapped instruments.

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Class N - Naked solid product

Small autoclaves suitable for the sterilization of **unwrapped solid instruments**.

With these machines you can not sterilize hollow instruments, porous or wrapped. Although it has a limitation in the type of instruments to be sterilized, the autoclave in class N has many strengths, such as the **smaller dimensions**, the **rapidity of the cycle**, **reliability** and of course the **lower price**.

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CLASS S - Specified by the manufacturer

It is a middle class, whose type of instruments to be sterilized must be specified by the manufacturer. **You can NOT sterilize all types of instruments**. **Many security in less**, slightly lower price, and in some cases, higher price than the *phoenix Blu*.

CLASSE S - Specified by the manufacturer

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CLASS S

SOLID INSTRUMENT

Tools, which can be subjected to sterilization, without cavities and without obstacles for the penetration of steam.

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POROUS MATERIALS

Porous bodies are simple or compound materials that can absorb fluids (tissues, gowns, etc.)

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HOLLOW INSTRUMENT

The hollow material are materials or instrument with cavities, obstructions, etc ...

Type A: turbines, handpieces and devices with blind holes or small

Type B: cannulas, tubes or devices with large passages

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OUR PRODUCTION

CLASS B AUTOCLAVES

According to the Directive UNI EN 13060, to which all our autoclaves are comply, Class B autoclaves are the only ones that can sterilize any kind of instrument. Autoclaves are fully automatic and with similar performance to the hospital autoclaves.

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CLASS N AUTOCLAVES

The autoclave class N is suitable to sterilizing only unwrapped solid instruments. Although it has a limitation in the type of instruments to be sterilized, the class N autoclave has several strengths, such as the smaller size, a faster sterilization cycle, reliability and of course the price.

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HOT AIR STERILIZER

The devices are hot air sterilizers that are commonly called "DRY HEATERS" and can be of two types, depending on the mechanism of air circulation inside the devices.

- 1) Natural air circulation
- 2) Forced air circulation

The operation for both types, is based on heating, with electrical resistances, of the air. This circulates in the sterilization chamber and sterilizes the instruments at the required temperature.

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SEALER AND ACCESSORIES FOR STERILIZATION

The main products of our company are autoclaves and sterilizers.

To offer a complete service to our customers, over the years we have enriched our production to a variety of accessories: Thermo-sealer, distiller water, trays, baskets and boxes, bags and rolls for sterilization, Control Test for autoclaves and sterilizers.

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CLASS N AUTOCLAVE

HYDRA 100

**CLASS N AUTOCLAVE
COMPLY TO DIRECTIVE 93/42
MEDICAL DEVICES CLASS IIb**

A HYDRA 100 IS FOREVER

The average lifespan of an autoclave **HYDRA 100** is around 8 years although some of our clients still require spare parts for autoclaves more than 15 years old. (data result from a market research.)

Autoclave Hydra 100 is a Class N autoclave, fully automatic, easy to use and carry out complete cycles of sterilization at 125 ° C in just 15 minutes.

The main quality is reliability!

With little maintenance and the replacement of a few parts subject to wear this autoclave can continue to sterilize for more than 15 years.

Available in version 9 liters and is compliant with Directive 93/42/EEC on small steam sterilizers. With its reliability, speed, and less space is a good alternative to class B autoclaves for the sterilization of unwrapped solid instruments.

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HYDRA 100 AUTOMATIC

The autoclave HYDRA 100 is equipped by a heating element placed under the sterilization chamber which allows the heating to 125 ° C of the distilled water. The pressure that is created inside the sterilization chamber, in combination with the temperature, allow the complete asepsis on the instruments. The sterilizer is equipped with several sensors that monitor the reaching of the temperature, which is preset and can not be changed to prevent tampering.

The operation is very easy just rotate the knob that controls the time of the sterilization cycle and the autoclave begins the heating phase. Upon reaching 125 ° C a light signal will inform the start of the sterilization phase.

At the end of the cycle just open the vent valve and wait 2 minutes. The sterilization cycle is now complete and it is possible to unload the sterilized instruments.

The machine is equipped with several safety systems, in particular with a thermostat that, in the event of an excessive temperature, stops the sterilization cycle. It is also equipped with a safety valve with TUV certification (PED IV), the same installed on the class B autoclaves.

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HYDRA 100 AUTOMATIC

➤ THE MAIN QUALITY IS RELIABILITY!

Autoclave Hydra 100 is a Class N autoclave, is compliant with Directive 93/42/EEC on small steam sterilizers. With its reliability, speed, and less space is a good alternative to class B autoclaves for the sterilization of unwrapped solid instruments.

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DT since 1964
MEDICAL TRADING

MEDICAL TRADING S.R.L.
MANUFACTURER OF AUTOCLAVE SINCE EVER, FOREVER

BE SURE!

IN THIS CENTER ARE USED
STERILIZED INSTRUMENTS WITH AU
HYDRA 100

HYDRA 100