Manual de instrucciones  
Instruction manual  
Mode d'emploi  
Bedienungsanleitung  
Manuale d'istruzioni

Microdermoabrasión con puntas de diamante  
Dermabrasion  
Microdermabrasion par têtes a diamant  
Diamant microdermabrasionsgerät  
Microdermoabrasione con punte di diamante

---

## Advertencia

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## Warning

- Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.
- Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components of your unit.
- Never use the unit outdoors; exposure to the elements can cause damage.
- If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

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## EC-Declaration of compliance

This product meets the following requirements:

1. Conditions set out in the Low Voltage Directive: 2014/35/EU
2. Conditions set out in the Electromagnetic Compatibility Directive: 2014/30/EU

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## Recycling

Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve save natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.

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---

## Welcome!

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation.

The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

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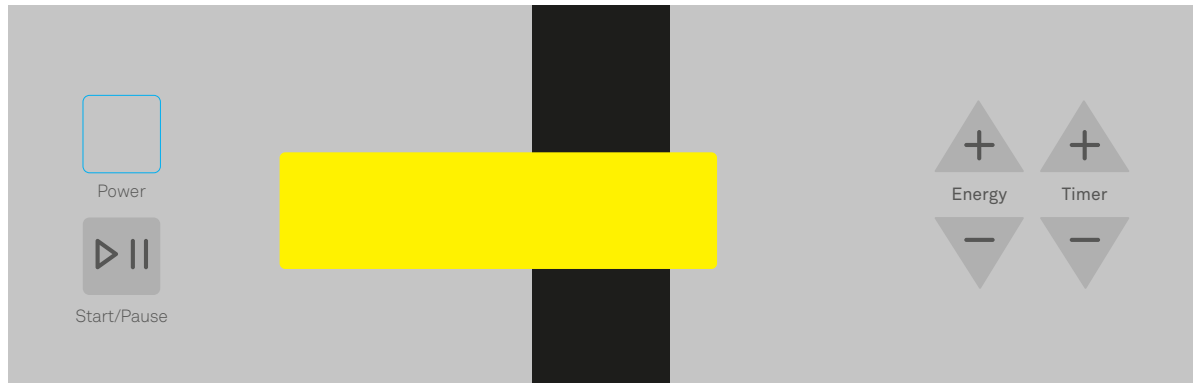
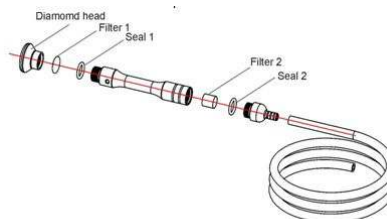
### Instructions of use

1. Connect the power supply via the rear panel.
2. Turn on the "POWER" button.
3. Correctly connect one side of the pipe to the rear panel as indicated in the sketch.
4. Connect the opposite side of the pipe to the vacuum handle. Place the correct diamond head in the handle. There are nine types of diamond heads.
5. Check that the foam filter is in its proper position in the handle.
6. Check that the metal filter is in its proper position in the handle.
7. You can adjust the suction effect by turning the regulator.
8. You can adjust the suction effect by covering or uncovering the hole in the side of the handle with your finger.
9. Once you have finished, first turn off the unit by pressing the "POWER" button.

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Ø 22 MM	
D100	Body. Suitable for acne, knees, elbows and areas with stretch marks
D180	
Ø 29 MM	
D180	Face. Suitable for large spots, pores, aging sun exposure
D220	
Ø 15 MM	
D100	Scars, fine lines and various imperfections
D200	
D250	
Ø 8 Y 6 MM	
D200	Eyes and both sides of the nose.
D250	Wrinkles, acne, pimples and blackheads

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## Cautions

- Do not leave the handle on the same part of skin for too long since it can cause injuries.
- Avoid using it on irritated skin.
- The following skin areas are unsuitable for these treatments: skin tumors, moles, birthmarks, angiomas, lips, eyes, herpes and warts.
- Do not work the vacuum pump for extended periods without cool-down time between treatments in order to prevent overheating of the pump and damage to the machine. Damage caused by overheating of the pump is not covered by the manufacturer's warranty.

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---

## Maintenance

- The foam filter in the handle needs to be changed after every 5 treatments. Dismantle the handle and replace the filter. Then reassemble the handle.
- The metal filter in the handle needs to be changed after every 5 treatments. Dismantle the handle and replace the filter. Then reassemble the handle.
- Clean and sterilize the diamond heads after every treatment.

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